EN Instructions for Use
Read these instructions for use before you obtain a blood drop from your fingertip.

Intended Use
Lancing Device
The Accu-Chek Softclix lancing device is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The Accu-Chek Softclix lancing device is intended for single patient use only! It is only allowed to be used to obtain blood from one person.

There is a risk of infections being transmitted if the lancing device is used by other people, even by family members, or if healthcare professionals use this lancing device to obtain blood from different people. This lancing device is therefore not suitable for professional use in healthcare facilities and institutions. It is not a sterile product.

People in the area of diabetes self-monitoring are considered as target group.

Lancet
The Accu-Chek Softclix lancet and the Accu-Chek Softclix XL lancet are sterile single-lancets and are intended to be used with the Accu-Chek Softclix lancing device for simple, virtually pain-free collection of capillary blood. The lancet is a disposable article intended to be used only once each.

Each lancet has a protective cap that guarantees sterile condition until its first use. The lancets are sterilized using irradiation.

General Safety Information
The instructions for use contain warnings and precautions.
A WARNING indicates a foreseeable serious hazard.
A PRECAUTION describes a measure you should take to use the product safely and effectively or to prevent damage to the product.

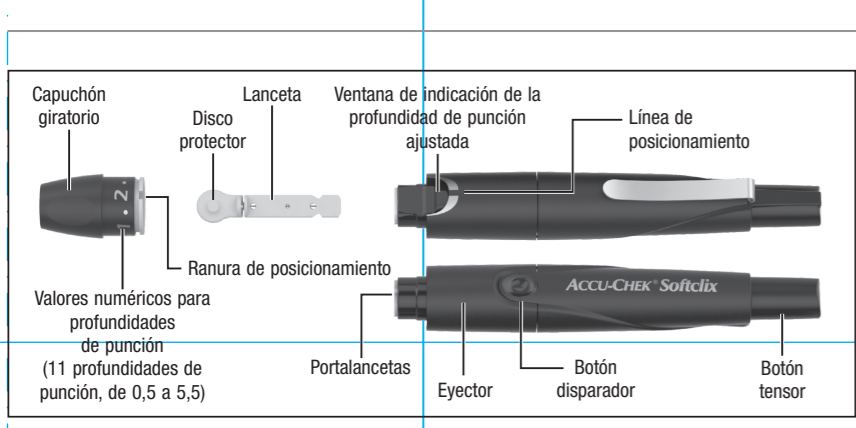
EN

WARNING
Risk of suffocation
This product contains small parts that can be swallowed.
Keep the small parts away from small children and people who might swallow small parts.

Risk of injury
If the penetration depth is too deep, the puncture can injure children.
Before using the lancing device for the first time on children, ask your healthcare professional. Start with the lowest penetration depth.

Risk of infection
If the lancing device or the lancet has come into contact with blood, it can transmit infections.
The lancing device and the lancet must not be used by other people, even by family members. Healthcare professionals must not use them to obtain blood from different people.
If the lancing device or the lancet was dropped or exposed to other mechanical stress, check it for damages. If you spot damages such as cracks, do not use the lancing device or the lancet and discard it.

PRECAUTION
Risk of injury
• Skin contact to the lancing device or lancet can cause skin irritation.
If your skin becomes irritated or inflamed, consult your healthcare professional.
• If the lancet is stabbed into a hard surface, the needle can be damaged.
Only use the lancet to obtain a blood drop.
• Parts of the lancing device, for example, the priming button, may shoot away during use.
Do not point the lancing device at your face or other people.

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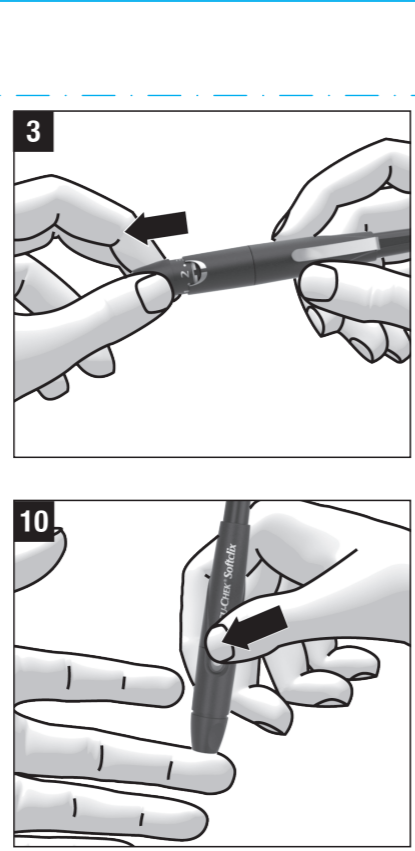
Only insert Accu-Chek Softclix lancets into the Accu-Chek Softclix lancing device. Other lancets can impair the operation of the lancing device or damage it permanently.

Tips and Tricks
If you have no experience with this lancing device, start with a low penetration depth, such as penetration depth 2.
When you press the priming button down, do not press the release button simultaneously.
The lancing device is primed when the center of the release button is yellow.
In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present. Obtain capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.
If not enough blood emerges, apply more pressure on the lancing device when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition. If too much blood emerges, decrease the penetration depth.

Using the Lancing Device

EN

WARNING
Risk of infection
If the lancing device or the cap has come into contact with blood, it can transmit infections.
Clean and disinfect the lancing device and the cap always before someone else handles them, for example, to assist you.

PRECAUTION
Risk of infection
If the lancing device or the cap has come into contact with blood, it can transmit infections.
Clean and disinfect the lancing device and the cap always before someone else handles them, for example, to assist you.

To prevent infections from being transmitted, you must clean and disinfect the lancing device and cap regularly:

- At least once a week.
- When there is blood on the lancing device or cap.
- Always before someone else handles the lancing device, for example, to assist you.

You need 3 lint-free cloths, water and 70 % isopropanol. The cloth used for disinfection must be large enough to wrap the lancing device in. Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the lancing device and the cap:

- Do not dip the lancing device and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.

► Remove the cap from the lancing device.
► Eject the lancet.
► For cleaning, moisten the first cloth with water.
► Wipe the lancing device and the cap thoroughly with the moist cloth. Wipe the lancing device from the outside only and the cap from the outside and the inside.

EN

PRECAUTION
Risk of infection
If the lancing device or the cap has come into contact with blood, it can transmit infections.
Clean and disinfect the lancing device and the cap always before someone else handles them, for example, to assist you.

To prevent infections from being transmitted, you must clean and disinfect the lancing device and cap regularly:

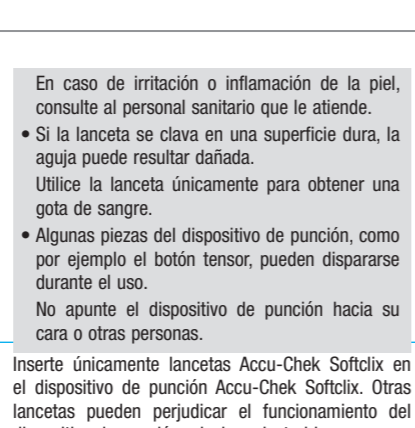
- At least once a week.
- When there is blood on the lancing device or cap.
- Always before someone else handles the lancing device, for example, to assist you.

You need 3 lint-free cloths, water and 70 % isopropanol. The cloth used for disinfection must be large enough to wrap the lancing device in. Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the lancing device and the cap:

- Do not dip the lancing device and the cap in the fluids.
- Use only water for cleaning and only 70 % isopropanol for disinfecting.

► Remove the cap from the lancing device.
► Eject the lancet.
► For cleaning, moisten the first cloth with water.
► Wipe the lancing device and the cap thoroughly with the moist cloth. Wipe the lancing device from the outside only and the cap from the outside and the inside.

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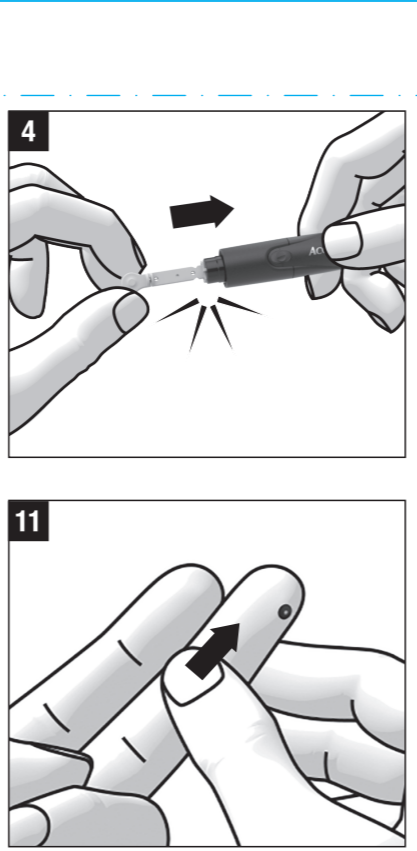
Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

Operating Conditions
Use the lancing device including accessories under the following operating conditions:
• Temperature range: +5 to +45 °C
• Relative humidity range: 5 to 90 %

Storing the Lancing Device and the Lancets
Do not store the lancing device in a primed state. Do not store the lancing device and the lancets at extreme temperatures, for example, in a hot car. This can impair the operation of the lancing device and the lancets.
Store the lancing device only without a lancet inserted.

Cleaning and Disinfecting the Lancing Device and the Cap

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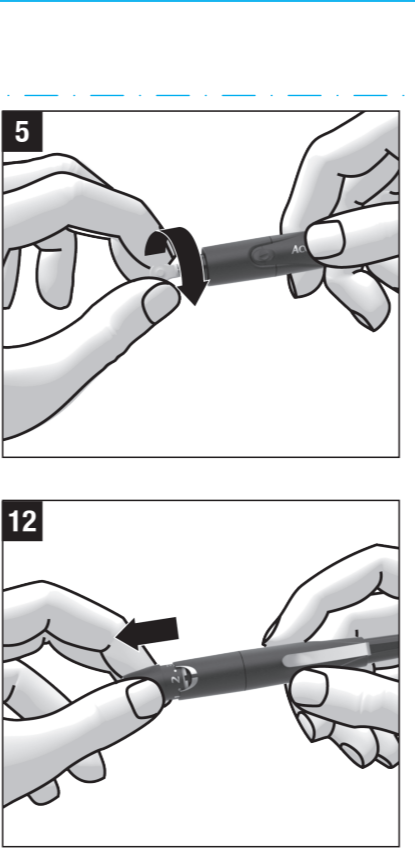
the outside only and the cap from the outside and the inside.

- Dry the lancing device and the cap with a second, dry cloth.
- For disinfection, dampen a third cloth with 70 % isopropanol.
- Wipe the lancing device and the cap thoroughly with the isopropanol cloth for 2 minutes each. Wipe the lancing device from the outside only and the cap from the outside and the inside.
- Place the cap back on again.
- Wrap the whole lancing device in the isopropanol cloth.
- Leave the lancing device wrapped for 8 minutes.
- Unwrap the lancing device and let it dry in the air.

Discarding the Used Lancet

EN

WARNING
Risk of infection
A used lancet can transmit infections.
The needle of a used lancet can be touched accidentally. Therefore, discard the lancet in such a way that the lancet does not cause injury to anyone. For example, place the lancet into a sharps container.

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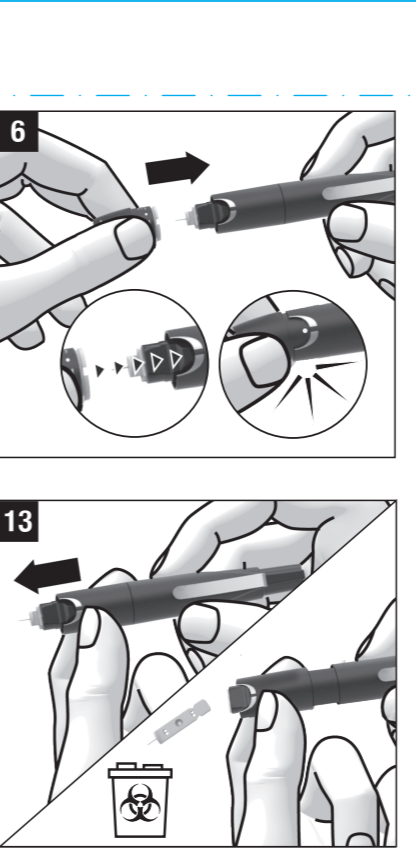
Customer Support
Roche Diabetes Care Australia Pty. Limited
2 Julius Avenue, North Ryde
NSW 2113, Australia
ABN 69 602 140 278
Accu-Chek Enquiry Line: 1800 251 816
www.accu-chek.com.au

Canada
Accu-Chek Customer Care: 1-800-363-7949
www.accu-chek.ca

EN

PRECAUTION
Risk of infection
If the lancing device has come into contact with blood, it can transmit infections.
Discard the used lancing device as infectious material according to the regulations applicable in your country.

Discard an unused lancing device as plastic waste. For information on how to discard the lancing device correctly, contact your local council or authority.

Ordering Information
If you want to order any of the Accu-Chek Softclix products, contact customer support (see section Customer Support).

Reporting of Serious Incidents
For a patient/user/third party in the European Union and in countries with identical regulatory regime; if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Last Update
2022-02

EN

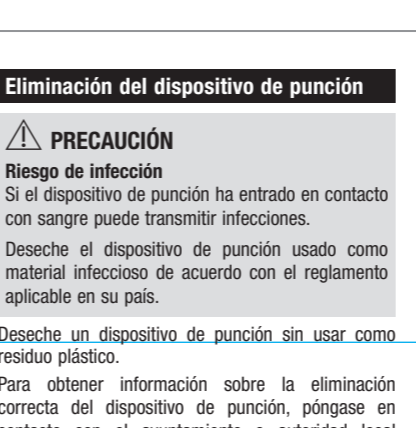
PRECAUTION
Risk of infection
If the lancing device has come into contact with blood, it can transmit infections.
Discard the used lancing device as infectious material according to the regulations applicable in your country.

Discard an unused lancing device as plastic waste. For information on how to discard the lancing device correctly, contact your local council or authority.

Ordering Information
If you want to order any of the Accu-Chek Softclix products, contact customer support (see section Customer Support).

Reporting of Serious Incidents
For a patient/user/third party in the European Union and in countries with identical regulatory regime; if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

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© 2022 Roche Diabetes Care
Roche Diabetes Care GmbH