
ACCU-FINE®

An ACCU-CHEK® product

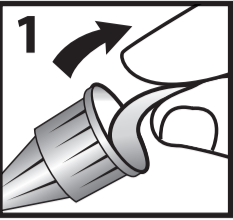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EN Instructions for use

- Remove the seal from the pen needle.
- Push the pen needle straight on the pen injector device and twist until it is tight (2a). Do not push the pen needle at an angle. Then remove the outer protective cap and set it aside (2b). Do not discard it. You will need it later to remove the pen needle from the pen injector device.
- Remove the inner protective cap by pulling it straight out to prevent bending of the needle and/or damaging the needle tip.
- Prime your pen injector device as per your pen's instructions for use to prevent incorrect dosage caused by air in the cartridge or a clogged needle. If no drug flow is seen after several attempts, attach a new pen needle.
- Set your drug dose.
- Make your injection as directed by your healthcare professional, do not bend the needle. Inject slowly. Count to 10 before removing the needle from the skin to deliver a complete dose and to prevent a drug leakage. Do not change the direction of the pen needle while it remains in the body as this can result in bending or breaking of the needle.
- Replace the outer protective cap.
- Pen needle is single-use only; remove the pen needle from the pen injector device after each injection to prevent air from entering the cartridge and drug from leaking out. Discard used pen needle into a sharps container.

Warning: Remove both the outer protective cap and the inner protective cap before an injection. If both the outer cap and the inner protective cap are not removed before use, the medication or dose may not be injected, which may result in serious injury or death.

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EN COMPATIBLE WITH MOST PEN INJECTORS AVAILABLE ON THE MARKET:

ACCU-FINE® pen needles are compatible with the following pen injectors available on most markets: Omnican® Pen 31/32-**B. Braun**; BerliPen® 301/302, Autopen® 24/ Classic-**Owen Mumford**; BerliPen® JUNIOR/ areo/ areo2/PRECISION, JuniorSTAR®, TactiPen®, i-pen, i-pen®-**Haselmeier**; GensuPen®-**Copernicus**; GensuPen 2, GensuPen IMPROVE-**Nemera**; Humalog®KwikPen, moujano®KwikPen, ABASAGLAR® KwikPen®, Humulin® N Pen, HumaPen® Ergo/ Ergo II/ L U X U R A/L U X U R A HD/savvio, βyetta®Pen, Forsteo®-**Eli Lilly**; FlexPen®, FlexTouch®, wegony®FlexTouch®, NovoLet®, NovoPen® 3/4/5/6/Echo®/Echo®Plus, Junior, Victoza®, Xultophy®, Ozempic®, Saxenda®, Levemir® InnoLet® - **Novo Nordisk**; CikSTAR®, SoloSTAR®, AllStar®PR0, Lyxumia® -**Sanofi-Aventis**; ypsilon®, Gansulin Pen, PolihumIn Pen, BioamatkPen®- **Ypsomed**; PuregonPen® - **BD Medical**; Xiu Lin Pen - **Gani Technology**; penqid 2.0 - **Diamesco**; inpen™ - **Companion Medical Inc.**; **ACCU-FINE® pen needles can be used with the following pen injectors available in the USA:** HumaPen® LUXURA HD, HumaPen® Ergo, HumaPen® Ergo II – **ELI LILLY**, Autopen® – **OWEN MUMFORD**, TactiPen® – **HASELMEIER** Find the detailed list of compatible pen injector devices on: www.htl-strefa.com

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EN

Sterile, single-use pen needles

Intended for use with pen injector devices for the subcutaneous injection of drugs. For healthcare professionals and lay users.

NOTE: Read the needle gauge and length on the seal to make sure you are using the right pen needle for you or ask for assistance if you have impaired vision. Also do so, if you are a healthcare professional, a caretaker or a family member assisting a patient.

How to get ready for injection:

Choose and disinfect the injection site. Change the injection site often to avoid tissue hardening. You can inject drug into the fatty tissue of the abdomen, buttocks or the outer side or front of the upper thigh. Injections into the upper arm or the inner side of the upper thigh are not recommended as the risk of injecting into a muscle is high.

Injection techniques: The injection site and technique should be recommended to you by your healthcare professional. **Recommendation: Injection with a skin fold at a 90° angle:** it is recommended that you inject into a skin fold vertically to prevent too deep injection and hitting the muscle tissue. To make a skin fold when injecting, pinch the skin between the thumb and forefinger and do not pull up the muscle. **Injection with a skin fold at a 45° angle:** this injection technique is only recommended when you have very little subcutaneous fatty tissue. The minimum angle should be 45°. If the angle is less than 45°, the drug may not reach the subcutaneous fatty tissue and result in swelling or bruising at the injection site. **Injection without a skin fold at a 90° angle:** if your layer of subcutaneous fatty tissue is very thick, you can make an injection without a skin fold. When you do not use a skin fold, it is recommended that you inject at a 90° angle and choose a short or medium-length needle (up to 6 mm).

See table below for suggested needle length per your age and BMI:

AGE	BMI	NEEDLE LENGTH*	
		Without skin fold straight injection	With skin fold angled or straight injection
Children	—	—	4mm, 5mm
Slim adults	—	—	4mm, 5mm, 6mm, 8 mm, 10mm
Adults	Average	4mm	5mm, 6mm, 8mm, 10mm, 12mm
	Overweight (BMI>25)	4mm, 5mm, 6mm	8mm, 10mm, 12mm

* Consult your healthcare professional for assistance in choosing the appropriate needle length, injection technique and site.

⚠ CAUTION:

Avoid infections. Do not re-use or share pen needles. Do not use pen needles if the individual packaging is damaged or if the seal is missing or if the needle appears to be damaged. To avoid accidental needlesticks, never handle someone else's pen needles. Pen needles are single-use only and should be disposed of safely in accordance with the local regulations. Keep out of reach of children without adult supervision.

Remember to rotate the injection site! Follow the instructions for use, specific storage and handling recommendations of your pen injector device and the patient information leaflet of the drug. Seek immediate medical attention if you have any health concern.

Any serious accident that has occurred in relation to the device should be reported to the manufacturer and the local competent authorities / government agencies, as applicable.

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